Rio II Nasal Pillows Interface User Manual

Rio II Nasal Pillows Interface

Thank you for choosing the **Rio II Nasal Pillows Interface**. This mask is designed to offer comfort, flexibility, and quiet operation. It minimizes facial contact, helping to ensure a comfortable therapy experience. This user manual provides you with the information you need for the correct use of your mask. The **Rio II Nasal Pillows Interface** and its accessories are not made with natural rubber latex or DEHP.

Intended use

The mask is intended to provide an interface for patients who have been prescribed CPAP or bi-level therapy. The mask is:

- To be used by adult patients (>66 lb/30 kg) for whom positive airway pressure therapy has been prescribed.
- Intended for single-patient re-use in the home environment and multi-patient re-use in the hospital/institutional environment.

Contraindications

This mask should be used with caution for patients with the following conditions:


- Facial deformity.
- Injured or compromised facial skin in areas that come into contact with the mask during use.

Before using the mask

⚠ WARNINGS

- The vent holes must remain undamaged and free of blockages.
- The mask is not suitable for life-support ventilation.
- Use only with CPAP or bi-level devices recommended by a physician or clinician.
- Do not use the mask unless the therapy device is powered and delivering airflow. Once the mask is fitted, confirm that the device is blowing air.
- Rebreathing warning:** CPAP and bi-level devices require masks with vent holes. When the device is turned on and operating properly, the airflow from the device flushes the exhaled air out through the exhalation port. However, when the device is not operating, insufficient fresh air may lead to rebreathing of exhaled CO₂, which can lead to suffocation.
- With low PAP pressures, the flow through the exhalation port may not be sufficient to clear all exhaled gas from the tube, increasing the risk of rebreathing.
- To minimize the risk of vomiting during sleep, patients should avoid eating or drinking, three hours before using the mask to minimize the risk of vomiting and aspiration. This mask is not recommended if the patient is taking medication that may induce vomiting.
- Do not use with patients allergic to silicone.
- This mask should not be used on patients who are uncooperative, unconscious, unresponsive, or unable to remove the mask.
- When the patient is unable to remove the mask himself, the mask must be used under qualified supervision.
- Using the mask may cause dental discomfort in the teeth, gums or jaw or aggravate pre-existing dental conditions. If these conditions occur, consult your health care professional.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bi-level device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in well ventilated rooms.
- At a fixed flow rate of supplemental oxygen, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.
- The technical specifications of the mask are provided for your clinician to check if they are compatible with the flow generator. If used outside of specifications or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the CPAP or bi-level function.
- Discontinue usage immediately and consult your physician or clinician, if you experience ANY adverse reactions.
- Refer to your flow generator manual for details on settings and operational information.
- Remove all the packaging before using the mask.
- Images shown here are indicative only. If there is inconsistency between the image and actual product, the actual product shall govern.
- Any serious incidents that occur in relation to this device should be reported to React Health.
- Use only specified cleaning agents and methods outlined in the accompanying documents.
- Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in the accompanying documents, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.

Components of the Rio II Nasal Pillows Interface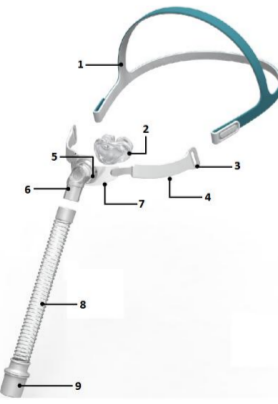


- | | |
|------------|--|
| 1 Headgear | 2 Nasal Pillows (available in S, M, L) |
| 3 Panel | 4 Soft sleeves |
| 5 Clip | 6 Elbow |
| 7 Frame | 8 Short tube |
| 9 Swivel | |

Fitting the mask

Notes:

- Refer to your therapy device manual for details on settings and operational information.
- Three nasal pillows sizes (S, M, L) are available. Gently insert the pillows into the nostrils to ensure the size (e.g. M) is correct for you.
- If you detect an air leak, gently adjust the angle of the nasal pillows to achieve a secure seal. If leaking persists, try a different pillow size.

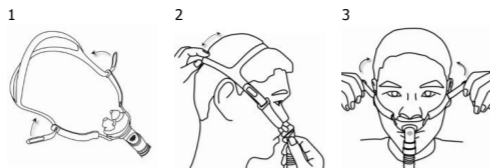Pass the side straps of the headgear through the installation slots of the panels. Fold and secure.

Place the nasal pillows under the nose and pull the headgear over the back of the head. Ensure the top strap sits comfortably on the top of the head.

Adjust the side headgear straps evenly, until the mask sits securely and comfortably under the nose.

Slightly take the mask away from your face and place it back to achieve a comfortable and effective seal. Ensure the nasal pillows fit securely into the nostrils with both pillows upright.

Finally connect the short tube with the therapy device-connected breathing tube via the swivel. Your mask is now fitted and ready to use.


Removal

To remove the mask, first gently pull the mask away from your face, then lift the headgear up and over your head. There is no need to loosen the headgear straps.

Disassembling the mask

Notes:


- The short tube assembly cannot be disassembled from the elbow.
- To disconnect, always grasp the connector itself—never pull on the tube. Pulling the tube can cause invisible damage that might lead to air leaks.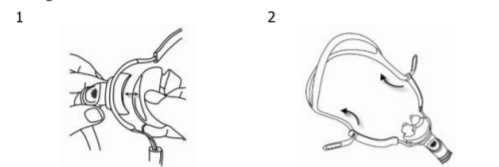

Remove the nasal pillows assembly from the frame assembly by holding its top and bottom and gently pulling it away from the frame.

Undo and gently slide the side headgear straps out of the slots of the panels.

Reassembling the mask

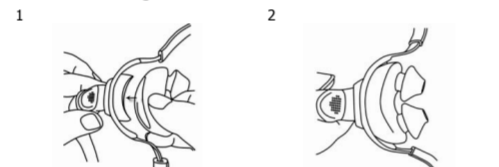Position the nasal pillows assembly onto the mask frame in the correct orientation.

Press firmly around the entire assembly to ensure a tight, gap-free seal.

Slide the side headgear straps in the slots of the panels and secure them.

Cleaning the mask at home

⚠ CAUTIONS

- Do not use cleaning solutions that contain bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils. These solutions may damage the mask and shorten its lifespan.
- Exposing any part of the mask to direct sunlight or heat may cause deterioration.
- Inspect all mask components regularly. Visual criteria for product inspection: If there are any signs of visible deterioration (cracking, crazing, tears, etc.), the affected part should be discarded and replaced.

Preparation before cleaning

- Gather cleaning supplies: soft bristle brush, drinking water (10 litres per mask), mild liquid detergent (e.g. mild liquid soap).
- Disassemble the mask components according to the disassembly instructions.

Daily / After each use: All mask components except headgear

- Make a solution of mild liquid detergent (e.g. 2% mild liquid soap diluted with drinking water, 25°C to 35°C), following the manufacturer's recommendation. The water temperature should be between 25°C to 35°C.
- Soak the components for 10 minutes.
- While soaking, use a soft bristle to clean each component thoroughly for at least 1 minute. Pay special attention to all crevices and cavities.
- Rinse the components by agitating them vigorously in 5 liters of clean drinking water. Repeat the rinsing step a second time.
- Place components on a clean, dry surface and allow to air dry out of direct sunlight.
- When all components are dry, reassemble the mask following the reassembly instructions.

Note:

- Clean daily or after each use.

Weekly: Headgear

Hand wash the headgear in a solution of mild liquid detergent (e.g. 2% mild liquid soap diluted with drinking water, 25°C to 35°C) for 10 minutes. Rinse the headgear well with drinking water and allow it to air dry out of direct sunlight.

Note:

- Clean once a week.

Reprocessing the mask between patients

This mask should be reprocessed when used between patients. Cleaning and disinfection instructions are available from the BMC website, www.bmc-medical.com. If you do not have internet access, please contact your homecare provider.

Note:

- The maximum number of disinfection cycles is 20.

Intended clinical benefits

- Improve patients' quality of sleep.
- Improve patients' compliance with therapy.

Troubleshooting

Problem	Possible reason	Possible solution
Mask is uncomfortable.	Mask fitted incorrectly.	Reposition the mask on your face. Adjust the headgear straps evenly. Do not overtighten the straps.
	Mask size is incorrect.	Consult your clinician.
Mask is too noisy.	The noise is coming from the device, humidifier or circuit.	Confirm the source of the noise. See troubleshooting for mask leaks.
	Vent holes may be partially blocked.	Use a soft bristle brush to clean the vent holes.
	Headgear straps adjustment.	Re-adjust evenly. Ensure that the straps are not too tight.
Mask leaks around the face.	The cushion is not well connected onto the frame.	Check the insertion of the cushion and reinsert correctly onto the frame.
	The cushion is not positioned correctly.	Reposition the mask on your face.
	The cushion is creased.	Slightly adjust the position of the mask by reseating it. The cushion must sit evenly on your face for optimal comfort.
	The cushion is dirty.	Clean mask as per manufacturer's recommendations.
	Mask size is incorrect.	Consult your clinician for a mask refit.
	The mask, cushion or headgear is at the end of their service life.	Inspect and change as per manufacturer's recommendations or sooner.

Storage

Ensure that the mask is thoroughly clean before storing it for any length of time. Store the mask in a dry place out of direct sunlight.


Disposal

The mask does not contain any hazardous substances and may be disposed of with your normal household waste. For patients with transmissible diseases, the product shall be discarded as medical waste.

Service life

The service life of the mask depends on the intensity of usage, maintenance, and environmental conditions by which the mask is used or stored. As this mask system and its components are modular in nature, it is recommended that the user maintain and inspect it on a regular basis and replace the mask system or any components if deemed necessary or according to the "Visual criteria for product inspection" in the "Cleaning the mask at home" section of this manual. The expected service life of the Rio II Nasal Pillows Interface is one year from the date of first use.

Technical specifications

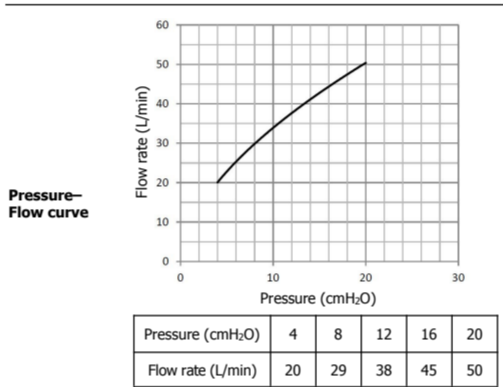Dead space information	Dead space is the empty volume of the nasal pillows and short tube assembly up to the swivel. The dead space of the mask for Large pillows is 92 mL.
Therapy pressure	4 to 20 cmH ₂ O
Resistance	Drop in pressure measured at 50 L/min: 0.5 cmH ₂ O; at 100 L/min: 1.6 cmH ₂ O
Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 26 dBA, with an uncertainty of 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 18 dBA, with an uncertainty of 3 dBA.
Environmental conditions	Operating temperature: +5°C to +40°C (41°F to 104°F) Operating humidity: 0% to 93% relative humidity non-condensing Storage and transport temperature: -20°C to +55°C (-4°F to 131°F) Storage and transport humidity: 0% to 93% relative humidity non-condensing
Date of manufacture	See product label
Use-by date	See product label

Symbols

System and packaging

	Caution		Use-by date
	Consult instructions for use		Not made with natural rubber latex
	Batch code		Model Number
	Temperature limit		Made in China
	Humidity limitation		Medical device
	Manufacturer		Catalogue number
	Importer		

Limited warranty

It is warranted that Rio II Nasal Pillows Interface, including its components, frame, nasal pillows, headgear, short tube and other accessories shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer. It is not transferable. To exercise the rights under this warranty, contact your local authorized dealers.

IMPORTER:
REACT HEALTH
110 - 2195, Chemin du Village
Mont-Tremblant, QC J8E 3M3
(888) 622-6515
www.reacthealth.ca

MANUFACTURER:
BMC MEDICAL CO., LTD.
Room 110 Tower A Fengyu Building, No. 115 Fucheng Road, Haidian, 100036 Beijing, PEOPLE'S REPUBLIC OF CHINA
URL: en.bmc-medical.com
E-mail: intl@bmc-medical.com
Tel: +86-10-51663880

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